




## Age Prevention Pico Wave

With young, fresh, healthy skin Neurotris advanced micro-current treatments can slow the aging process, help maintain a youthful glow, deliver hydration balance and support cellular metabolism. The following treatments are based on skin types. If your client has minimal dysfunction like clogged pores or a few blemishes these treatment steps will apply.

- 🌿 Pre-Cleanse to remove cosmetics if applicable.
- 🌿 2<sup>nd</sup> Cleanse using water base foaming cleanser and utilize your Photon Probe to deep pore cleanse in the following manner.
  - Apply Positive Clip Cable with Clip to upper arm, insert Photon Probe.
  - Set to Pico Refine
  - Set amplitude to client comfort,
  - Set Ultra-Sonic on high for normal to dry skin and low for combination to oily skin.
  - Set LED to Green for normal to dry Skin or Blue for oily to combination Skin. Use circular motions for 2-3 minutes and remove cleanser. Tone if desired.
- 🌿 Gently exfoliate with enzymes, soft scrubs or roto brush.
- 🌿 Using Infinity Probes large on black and small on red, apply Neurotris Conducting Gel,
  - Set Nano Refine and adjust amplitude to client comfort. With probes set to 1 to 2 inches apart begin on the right

and gently glide the probes across the neck and up to the chin, continue to glide in scissor like fashion over mouth, around the cheeks. Resting under the eye use the positive probe in circular motions in the corners of the nose and then continue gliding around the eyes on the full forehead and down the left side of the face in the same way. Eyes, nose mouth, chin and neck. This gentle gliding should take 3-5 minutes for the full face and neck in this setting. Take care not to touch the tips of the Infinity probes and remember to keep the black above the red.

 Spritz Neurotris Hydrating mist to rehydrate. Apply Neurotris conducting gel.

- Set to Pico Refine adjusting amplitude to client comfort. Using the infinity probes Large on black and small on red, focus on lips. Begin at the lower lip by setting the negative probe on lower lip fullness, pinching the two probes with the lip between them concentrate on the perimeter of the lip holding 5-8 seconds as you go around the mouth for 2-3 minutes.
- Then move to the nasal lip folds with negative above positive lift and hold the mouth lines for 8-10 seconds do this on both sides of the mouth for 2-3 minutes.
- Keeping the negative about positive use the erase movement slowly over upper lip and on lip folds around the entire mouth. Do this for 1-2 minutes.
- Continue to the eye area holding the negative probe on the upper orbital bone, with circular motions apply the positive probe on the lower orbital bone and erase

from the outside of the eye inwards for 1-3 minutes on each eye and in between the eyes on the glabella folds. Adjust amplitude to client comfort.

- 🌀 Refresh Neurotris conducting gel on face and switch to Conducting Gloves using the clip cables.
  - Set to Tone beginning at the neck on the right use gentle gliding motions with hold covering as much of the skin as possible taking care not to touch the gloves to one another. Using a combination of lift, glide and hold for 8-10 seconds at a time for each area of the neck, chin, cheeks, and forehead. Take 5-8 minutes for each side of the face.
- 🌀 Remove any traces of Neurotris gel and choose resting mask, for skin type. (cream, gel or clay) remove when finished working.
- 🌀 Apply water-based serums allow to absorb with a small amount of Neurotris conductive gel penetrate product further by using Photon Probe with positive Infinity probe and small wand.
  - Set Ultrasonic to low, Set Led to Green on the Ion Lift setting, adjust amplitude and use slow circular motions for 2-3 minutes covering the face and neck. Remove any conducting gel residue.
- 🌀 Finish with any additional serum, eye care, lip care, spf for day or nutrition/treat creams if evening appointment.